

ARCORA® GROUP

HYGIENE PROTECTION



ARCORA® GROUP


DISINFECTANT



HANDS



SURFACE

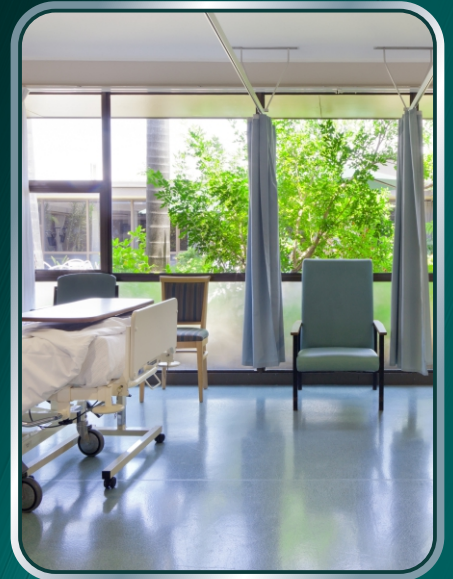

RAPID

DISINFECTANT

HAND DISINFECTION

AGAINST VIRUSES, BACTERIA, FUNGI, SPORES

pH 7



WITH ACTIVE CHLORINE

ARCORA HAND DISINFECTION quickly removes all bacteria, fungi, germs, viruses, moulds, spores and odours and does not allow for resistance and mutations. It contains activated and ionized components of water and salt and is non-toxic, non-irritating, **environmentally friendly** and highly efficient. Limited virucidal: Covered viruses (ECBO). **Slightly chlorinated odour.** Very good skin and mucous membrane compatibility (dermatological test successfully approved with "very good"). Can be applied in form of liquid, mist or ice. The Product is certified. **BAuA registration number: N-90755.**

**Tested according to EN1500 / 30 sec.,
EN13704 - EN1624 - EN13727 / 60 sec.**

- for hygienic hand disinfection
- does not cause allergies and is not carcinogenic
- does not leave incompatible residues
- no dry or cracked skin due to frequent use, no pain by contact with wounds or skin changes



content	article-no.	units
1 l bottle	01040	12x1 l
5 l canister	01056	1x5 l
10 l canister	01057	1x10 l

SURFACE DISINFECTION

AGAINST VIRUSES, BACTERIA, FUNGI, SPORES

pH 7



WITH ACTIVE CHLORINE

ARCORA SURFACE DISINFECTION quickly removes all bacteria, fungi, germs, viruses, moulds, spores and odours and does not allow for resistance and mutations. It contains activated and ionized components of water and salt and is non-toxic, non-irritating, **environmentally friendly** and highly efficient. Limited virucidal: Enclosed viruses (ECBO). **Slightly chlorinated odour.** Very good skin and mucous membrane compatibility (dermatological test successfully approved with "very good"). Can be applied in form of liquid, mist or ice. The Product is certified. **BAuA registration number: N-90753.**

Tested according to EN13727, EN13624, EN13697, EN13704, EN14476.

- suitable for use on all washable floor coverings and surfaces in areas at risk of infection, such as hospitals, nursing homes, sanatoriums etc.
- water, pipe, tank, surface, equipment and bottle disinfection, stable disinfection, and feather disinfection, hoof and claw care.
- agriculture: intermediate cluster disinfection, odour neutraliser for animal bowls, toys, toilets, toilet facilities, etc.
- avoid direct contact with food and animals.
- easy to use, time-saving and effective



content	article-no.	units
1 l bottle	01042	12x1 l
5 l canister	01059	1x5 l
10 l canister	01048	1x10 l

RAPID DISINFECTION

AGAINST VIRUSES, BACTERIA, FUNGI, SPORES

pH 7



WITH ACTIVE CHLORINE

ARCORA RAPID DISINFECTION quickly removes all bacteria, fungi, germs, viruses, moulds, spores and odours and does not allow for resistance and mutations. It contains activated and ionized components of water and salt and is non-toxic, non-irritating, **environmentally friendly** and highly efficient. Limited virucidal: Enclosed viruses (ECBO). **Slightly chlorinated odour.** Very good skin and mucous membrane compatibility (dermatological test successfully approved with "very good"). Can be applied in form of liquid, mist or ice. The Product is certified. **BAuA registration number: N-90756.**

Tested according to EN13727, EN13624, EN13697, EN14476

- suitable for use on all washable floor coverings and surfaces in areas at risk of infection, such as hospitals, old people's homes, sanatoriums, baths, saunas etc.
- water, pipe, tank, surface, equipment and bottle disinfection, stable disinfection, and feather disinfection, hoof and claw care
- agriculture: intermediate cluster disinfection, odour neutraliser for animal bowls, toys, toilets, toilet facilities, etc.
- avoid direct contact with food and animals
- easy to use, time-saving and effective



content	article-no.	units
1 l bottle	01052	12x1 l
5 l canister	01063	1x5 l
10 l canister	01064	1x10 l

ARCORA® GROUP

PROTECTIVE MASKS



SINGLE USE



FFP2



SHIELD

MASKS | FACE SHIELD

DISPOSABLE MASKS



With integrated metal clip, which adapts to the shape of the nose.

The breathable disposable masks 3-layer protect against droplets, dust, pollen etc. With elastic bands to fix behind the ear.

- 3-layer
- made of non-woven fabric
- certified and approved

article-no.	units
01084	50 pcs



FFP2 MASKS



With metal clip that adapts to the shape of the nose.

The tight fitting FFP2 masks protect against viruses and bacteria.

They are mainly used to protect oneself and others when the immune system is weakened, as well as to reduce the risk of infection during pandemics and flu epidemics.

- no exhalation valve
- made of non-woven fabric
- very comfortable to wear
- certified according to EN 149:2001 + A1:2009

article-no.	units
01085	5 pcs



FACE SHIELD

Comfortable to wear, flexible anti-fog face shield with elegant design in two different variants

- protects the whole face from cough, sneeze and saliva
- universally adjustable, for both women and men
- adjustable size thanks to adjustable closure
- rotatable by 120 degrees, if no protection is necessary
- can be worn over glasses
- easy to remove and replace protective PET film
- suitable for factory assemblies, public meetings, security personnel, supermarket employees, health care, hotels, restaurants, catering, airports, warehouse logistics etc.



Face shield with snapback closure

article-no.	units
01083	50 set

Contained in the set:
1 piece of anti-fog face protection film,
1 piece holding frame



Face shield 5IN1 with rubber band closure and cord stopper

article-no.	units
01123	60 set

Contained in the set:
5 pieces of anti-fog face protection film,
1 piece holding frame

ARCORA® GROUP

PROTECTIVE SUIT



PROTECTIVE SUIT TYPE 5/6



The protective suit type 5/6 of protective clothing category III offers protection when working in agriculture, environmental remediation, automotive industry, activities with paints and varnishes, chemical industry, construction industry, industrial cleaning, mineral fibre work, waste disposal, asbestos disposal, pharmaceutical industry, food industry, decontamination work, paint production, veterinary medicine.

Due to continuous fibres, which are spun with high density, the coverall offers a high barrier against fine fibres and particles and thus protects against external dangers. The material and structure allow air to pass through to the outside.

The protective jumpsuit is worn over regular work clothing. Thus the suit has a generous cut, which enables a lot of freedom of movement. An ergonomic cut with excellent wearability.

In addition, the suit is equipped with a 2-way-zipper with cover, as well as elastic incorporated rubber bands on sleeves, waist and ankles for a perfect fit.

size	article-no.	units
XXL	01090-XXL	1 pcs



Type 5 (EN ISO 13982-1)
Protection against solid or floating particles penetration.



Type 6 (EN 13034)
Protection against splashes and mists containing liquid chemicals.



EN 1149-5
Material provides electrostatic dissipation when properly grounded.



EN 1073-2
Protection against biologically contaminated liquids.



EN 14126
Protection against solid radioactive particles penetration.



ARCORA® GROUP

DISPENSER SYSTEMS



PROFI DESK DISPENSER



DISPLAY



PROFI EURO SPENSER

DISPENSER SYSTEMS AND DISPLAY

PROFI DESK DISPENSER 1000 ml



The high-quality **PROFI DESK DISPENSER** can be mounted on walls, or placed on tables or on another surfaces, if wall mounting is not possible or reasonable.

- 1000 ml bottle included in delivery
- with sturdy chromed metal housing and chromed metal operating lever
- hygienic handling with forearm possible
- bottle is easy to change
- compatible with ARCORA display stand for disinfection
- delivered ready for installation

size	article-no.	units
9,5x9,5x28 cm	01108	20 pcs



PROFI EURO SPENSER 1000 ml



The **PROFI EURO SPENSER** for walls is made of an aluminium housing with operating lever and nozzle made of stainless steel.

- 1000 ml bottle included in delivery
- hygienic handling with forearm possible
- bottle for refilling (does not comply with Euronorm)
- compatible with ARCORA display stand for disinfection
- delivered ready for installation

size	article-no.	units
16x10x29 cm	01093	20 pcs



DISINFECTANT STAND



The **Disinfectant stand** can be placed in any desired location. Ideal for entrance or highly frequented areas.

- compatible with PROFI EURO SPENSER or PROFI DESK Dispenser (1000 ml)
- quick and easy to set up
- printed with instructions for hand disinfection
- weight of stand: about 12 kg without dispenser

size	article-no.	units
45x50x155 cm	01115	1 pcs

Supplied without dispenser,
dispenser according to selection



ARCORA® GROUP

X-LINE WIPES



MOIST WIPING SYSTEM

X-LINE WIPES MOISTENING SYSTEM

WET WIPE DISPENSER INCLUDING CLOTH ROLL AS SET



**With absorbent, tear-resistant
and lint-free cloth roll**

The X-LINE WIPES MOISTURIZING SYSTEM is a refillable dispensing system for soaking wipes with disinfectants or cleaning agents.

Ideal for rapid and easy disinfection or cleaning wipe-resistant surfaces in medical facilities, schools, kindergartens, retirement homes, fitness centers, as well as in food processing, industrial and public facilities. Especially for areas that need to be disinfected or cleaned several times a day. Also ideally suited for hand disinfection.

The resealable dispenser bucket can be used universally and allows safe and easy removal. The closed system protects the liquid from contamination and evaporation.

- for all wipe-resistant surfaces
- easy handling
- universally applicable
- refillable system
- safe and user-friendly system
- economical use of wipes and application solution
- durability after impregnation - 28 days

dispenser size	article-no.	units
19,5x19,5x22 cm	01109	4 sets

Included in delivery with article no. 01109:

1 wet wipe dispenser volume 5 litres

1 white cover and 1 cloth roll with 350 sheets



PREPARATION

1. Put on gloves to protect the cloth roll from contamination.
2. Remove the cloth roll with gloves from its package.
3. Place the cloth roll in and pull out the first cloth.
4. Dose approx. 2.5-3 litres of ready-to-use application liquid in the desired concentration or use ready-to-use surface disinfectant.
It is recommended to use distilled or demineralized water for dilution to prevent streaks on the surface.
5. Fill in disinfectant solution in the desired application concentration.
6. Insert the first cloth through the cover opening and close the dispenser with the cover
Then wait 30-60 minutes so that the wipes can be soaked.
7. Mark the security label and stick it on to note the expiry date.
8. The wipes can be used for 28 days after soaking.

APPLICATION

Wipe objects and surfaces carefully until completely wetted.

Pay attention to the disinfectant's exposure time and concentration.

REFILLING

1. Put on gloves to protect the cloth roll from contamination.
2. Before refilling, dispose cloth and liquid residues.
3. Clean new or empty dispenser with alcoholic detergent.
4. Disinfect the dispenser with a cloth soaked in disinfectant for 1 minute.
5. Allow all parts to dry thoroughly and refill..

X-LINE SPARE DISPENSER BOX & CLOTH ROLL

SPARE DISPENSER BOX

The dispenser box made of robust plastic is ideal for impregnating wipes with disinfectant or detergent. With a tightly closable cover, which ensures an all-round airtight seal. Very suitable as a refillable dispenser system.

- with carrying handle for easy, fast and comfortable transport
- separately closable removal opening integrated in the cover
- wipes do not dry out
- hygienic extraction

dispenser size	article-no.	units
19,5x19,5x22 cm	01117	1 pcs

Included in the scope of delivery with article no. 01117 :
1 wet wipe dispenser volume 5 litres
1 white cover



SPARE CLOTH ROLL

A cloth roll with tear-off wipes for refilling the refillable dispenser system for impregnation with disinfectants or cleaning agents.

Ideal for areas where a high level of hygiene is required such as medical facilities, nursing homes, food industries etc. The durable wipes allow safe release of the disinfectant to the surface that is to be disinfected and ensure optimum surface wetting.

- 350 sheets, sheet size: 18x24 cm, roll size: 16.5x18 cm
- absorbent, tear-resistant and lint-free cloth roll
- easy to take out
- easy to use
- hygienic single use
- for professional use
- material: cellulose 60 %, polypropylene PP: 25 %, viscose 15

roll size	article-no.	units
16,5x18 cm	01116	16 pcs

Included in delivery with article no. 01116:
1 cloth roll 350 sheets

Cloth with two different sides:

- wiping side: glides optimally over the surface
- adhesive side: laminated side on which the hand lies


→ Due to lamination and disinfectant
no germ transmission



**NO
GERM TRANSMISSION!**

ARCORA® GROUP

AIRLAID-DISPOSABLE TOWELS

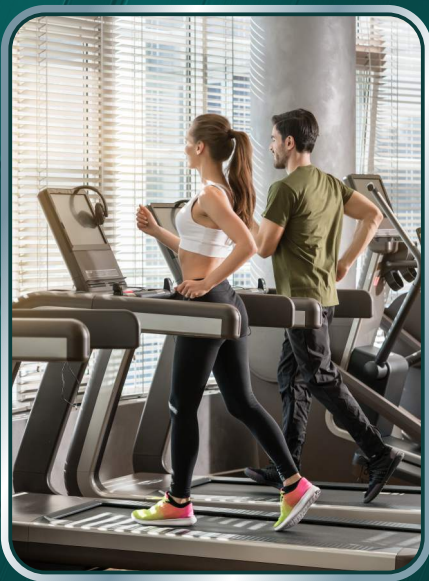

AIRLAID-DISPOSABLE TOWELS



Absorbent Airlaid disposable towels for single use made of environmentally friendly cellulose airlaid non-woven fabric from ARCORA.

It doesn't matter if it's for washing hair, drying hands or face, the use of towel is a regular routine. In hairdressing or beauty salons, a large quantity of dirty and used towels comes together daily. In addition, classic towels are a collection point for bacteria and germs.


Enough of this now!

The lint-free Airlaid disposable towels, which can be easily disposed are ideal for use in salons, hotels or fitness centres and thus replace conventional towels. An enormous absorbency, **400 times** its own weight, characterizes the antibacterial and hygienic Airlaid disposable towels.

- Hygienic protection against germs and bacteria
- Comfort and a safe feeling with the customer
- High absorbency
- Lint-free and efficient
- Soft and smooth
- Distinguishes itself from similar products by its weight (**55 g**)
- Material: 60 % cellulose, 30 % cotton, 10 % tree resin



sheet size	no. of sheets	colour	article-no.	units
50x75 cm	50 pcs	white	01122	5 rolls

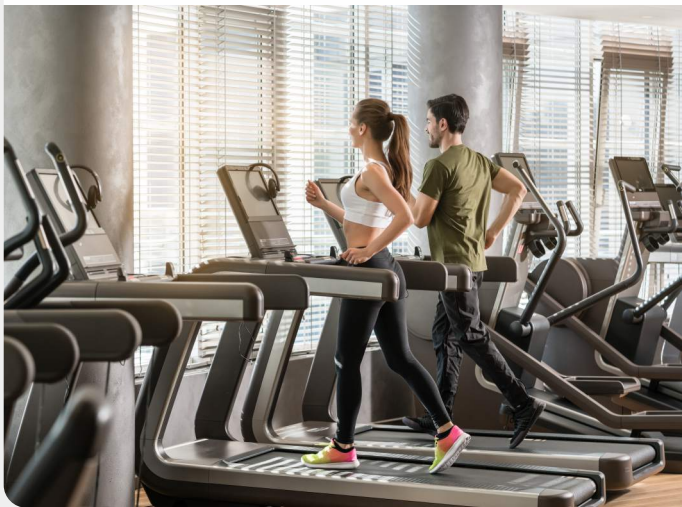

YOUR LOCAL DISTRIBUTOR



HEADQUARTER

ARCORA® INTERNATIONAL GmbH

Marsstrasse 9
85609 Aschheim near Munich / Germany

Telephone: +49 (0)89/ 14 33 29 3 - 0
Telefax: +49 (0)89/ 14 33 29 3 - 29

E-Mail: info@arcora.de
Web: www.arcora-group.com

ARCORA®
GERMANY

Visit us online at **www.arcora-group.com**. On the ARCORA® website you'll find the current product lines and their range of applications as well as a variety of useful information and a comprehensive range of services.